

ASM

Medicare

Picow Farm Road Ind Estate
Runcorn, Cheshire. WA7 4UG.

Tel: 01928 574301


Fax: 01928 575130

E-mail: sales@asmmedicare.com



Kingfisher Shower Stool

Code: 02014.

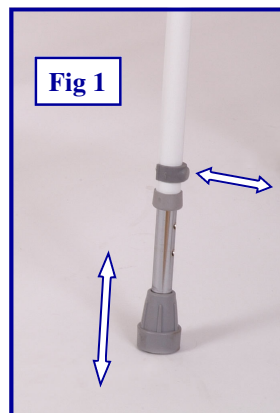

On receipt of your new ASM Kingfisher Shower Stool please check the contents for signs of damage, report any defects to **ASM Medicare Ltd** immediately. The stool comes fully assembled and ready for use, it is also height adjustable, to adjust please follow the instructions below.

- 1} Lay the frame on it's side and remove the plastic clips from 2 legs. See Fig 1.
- 2} Slide the internal legs to the required position ensuring that both legs are adjusted equally.
- 3} Replace the plastic clips, you may have to twist the leg slightly to allow the steel pin to locate completely, it is "Very Important" that the clips are fully engaged.
- 4} Turn the frame over and repeat for the other 2 remaining legs.
- 5} Place the stool back on it's feet on a level surface and check that it is stable and does not rock, if it does, readjust until it is stable.
- 6} Periodic checks of *all* screws and fittings is recommended, replace if required.
- 7} The plastic removable seat simply clips off and on by holding the stool and pulling away from the frame, replacing may require a slight tap with your hand. "Do Not" use a heavy object as this may cause damage to the removable plastic seat.
- 8} Clean the Kingfisher Shower Stool by using a mild proprietary soap and / or disinfectant. Dry with a soft clean cloth.

The intended use of this product is to provide the user with a height adjustable support stool, for use in a showering area.

M.U.W = 152kg / 24st.

NOTE:- These products are only to be used for their intended purpose.

For details of all our products please visit our web site at www.asmmedicare.com

Better By Design

User Instructions:
ASM Kingfisher Shower Stool.

IMPORTANT NOTICE!

Please read & fully understand before using or allowing others to do so.



June 2006