

ASM

User Instructions
ASM Chemic Commode 02143.



IMPORTANT NOTICE

Please read & fully understand before using or allowing others to do so.



October 2005.

Medicare

Picow Farm Road Ind Estate
Runcorn, Cheshire. WA7 4UG.
Tel: 01928 574301
Fax: 01928 575130
E-mail: sales@asmmedicare.com



Fig 2

Chemic Commode

Code: 02143

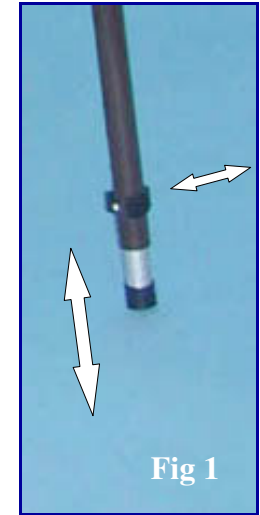


Fig 1

On receipt of your new ASM Chemic Commode please check for signs of damage, report any defects to **ASM Medicare Ltd** immediately. The ASM Chemic Commode comes fully assembled and ready for use. The commode is height adjustable, to adjust please follow the instructions below.

- 1} Place the commode on it's side and remove the plastic clips from 2 legs. See Fig 1.
- 2} Slide the adjustable internal legs to the required position ensuring that both legs are adjusted equally. See Fig 1.
- 3} Replace the plastic clips, ensuring that they are **fully engaged**. You may have to twist the leg slightly to locate the steel pin correctly.
- 4} Turn the commode over & repeat for the other two adjustable legs.
- 5} Place the commode back on it's feet on a level surface & check that it is stable and does not rock, if it does readjust the legs until it is stable. **ALWAYS ensure that the toilet seat is in the down position and sat on when toileting. See Fig 2.**
- 6} Periodic checks of all screws and fittings is recommended, replace if required.
- 7} to remove the bucket simply lift the seat and using the handle lift out the bucket, do not allow the bucket to become over full as this may lead to spillage or back injury due to the excessive weight of the bucket.
- 8} To clean the commode, a mild proprietary soap and / or disinfectant may be used. After cleaning dry with a soft clean cloth.

The Maximum User Weight for this product is: **190kg / 30st.**

The intended use of this product is to provide a portable temporary toilet and should not be used for any other purpose. The PVC cover is only intended as a cover and must not be used as a seat at any time.

NOTE:- These products are only to be used for their intended purpose.
For details of all our products please visit our web site at www.asmmedicare.com