

# ASM

ASM  
Sutton Mobile  
Shower Cradle  
User Instructions.

# Medicare

Picow Farm Road Ind Est  
Runcorn. Cheshire.  
WA7 4UG.

Tel: 01928 574301.

Fax: 01928 575130.

e-mail: [sales@asmmedicare.com](mailto:sales@asmmedicare.com)

## ASM Sutton Mobile Shower Cradle.

Code No: 02040S {small, colour code red}.

Code No: 02040M {medium, colour code yellow}.

Code No: 02040L {large, colour code green}.

A range of optional extras is available for these products,  
please contact our sales team for further information.

Model Shown:  
02040L



On receipt of your new **ASM** Sutton Mobile Shower Cradle please check for signs of damage, report any defects to ASM immediately. The shower cradle comes fully assembled and ready for use.

The intended use of this product is to provide a profiled mobile showering facility for one person.

Please read the following instructions before use.

- 1} Before placing a person in the cradle or removing them out all four castors **must** be locked.
- 2} When moving the cradle with a person in it please ensure that the lap strap is securely fastened.
- 3} The removable mesh cover should be periodically removed and inspected for signs of wear, the frame should also be inspected for wear or corrosion, we recommend this is done at three monthly intervals.
- 4} Before replacing the mesh on the frame we also recommend that the frame is coated with a good quality automotive wax. This will help to reduce the possibility of corrosion and prolong the life of the cradle.
- 5} If any wear or corrosion is found during inspection, remove the cradle from service and contact your representative or supplier for further advice.
- 6} To clean the cradle a mild proprietary soap and or disinfectant may be used, dry after cleaning with a soft cloth and coat the frame with automotive wax. The mesh can be machine washed following the washing instructions on the sewn on label using a **NON-BIOLOGICAL** soap powder. **Do Not Use Hot Air or Heat of Any Kind to Dry The Mesh!**

Maximum User Weight = 102kg / 16st.

**Note:** These products should only be used for their intended purpose.

For details of all our products please visit our web site at [www.asmmedicare.com](http://www.asmmedicare.com)



### IMPORTANT NOTICE!

Please read &  
fully understand  
before using or  
allowing others  
to do so.



Dec 2011



Polygiene®  
STAY FRESH