

ASM

Medicare

Picow Farm Road Ind Estate
Runcorn, Cheshire. WA7 4UG.
Tel: 01928 574301
Fax: 01928 575130
E-mail: sales@asmmedicare.com




Colne Toilet Surround.

Product Codes:

Chrome ~ 02049.

White ~ 02049W {shown}.

Fig 1

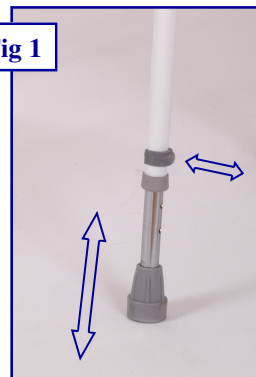

On receipt of your new ASM Colne Toilet Surround please check the contents for signs of damage, report any defects to **ASM Medicare Ltd** immediately. The Colne comes fully assembled and ready for use. The Colne is height adjustable, to adjust please follow the instructions below.

- 1} Lay the frame on it's side and remove the plastic clips from 2 legs. See Fig 1.
- 2} Slide the internal legs to the required position ensuring that both legs are adjusted equally.
- 3} Replace the plastic clips, you may have to twist the leg slightly to allow the steel pin to locate completely, it is "Very Important" that the clips are fully engaged.
- 4} Turn the frame over and repeat for the other 2 remaining legs.
- 5} Place the Cumbria back on it's feet on a level surface and check that it is stable and does not rock, if it does, readjust until it is stable. If you have purchased a floor fixing kit, it can now be secured in place using appropriate screws and plugs as required by your type of floor. These are not supplied and we also recommend that the fitting of the kit is carried out by a competent person or tradesman.
- 6} Periodic checks of *all* screws and fittings is recommended, replace if required.
- 7} Clean the Cumbria by using a mild proprietary soap and / or disinfectant. Dry with a soft clean cloth.

The intended use of this product is to provide the user with a height adjustable support aid, to fit around a standard toilet.

M.U.W = 127kg / 20st.

NOTE:- These products are only to be used for their intended purpose.

For details of all our products please visit our web site at www.asmmedicare.com

Better By Design

User Instructions:
Colne Toilet Surround.

IMPORTANT NOTICE!

Please read & fully understand before using or allowing others to do so.



October 2006