

# ASM

User  
Instructions  
ASM  
Canterbury  
Raised  
Toilet Frame.




## Medicare

Picow Farm Road Ind Estate  
Runcorn. Cheshire. WA7 4UG.  
Tel: 01928 574301  
Fax: 01928 575130  
E-mail: [sales@asmmedicare.com](mailto:sales@asmmedicare.com)

### Canterbury Raised Toilet Frame.

Code No: 02053.

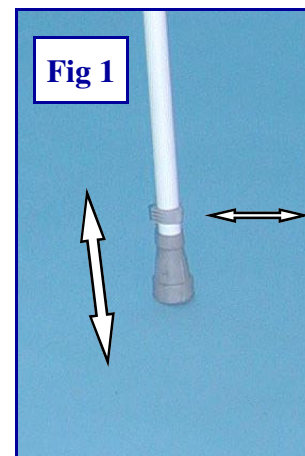

On receipt of your new ASM Canterbury Raised Toilet Frame please check the contents for signs of damage, report any defects to **ASM Medicare Ltd** immediately. The Canterbury comes fully assembled and ready for use. The Canterbury is height adjustable, to adjust please follow the instructions below.

- 1) Lay the frame on it's side and remove the plastic clips from 2 legs. See Fig 1.
- 2) Slide the internal legs to the required position ensuring that both legs are adjusted equally.
- 3) Replace the plastic clips, you may have to twist the leg slightly to allow the steel pin to locate completely, it is "*Essential*" that the clips are fully engaged to avoid injury.
- 4) Turn the frame over and repeat for the other 2 remaining legs.
- 5) Place the frame back on it's feet on a level surface and check that it is stable and does not rock, if it does, readjust until it is stable.
- 6) Periodic checks of *all* screws and fittings is recommended, replace if required.
- 7) Clean the Canterbury frame by using a mild proprietary soap and / or disinfectant. The plastic sleeve is also removable for ease of cleaning, it simply lifts out. Dry with a soft clean cloth.

**The intended use of this product is to provide the user with a height adjustable toilet seat, which can be used over an existing toilet without modification.**

**M.U.W = 152kg / 24st.**

**NOTE:- These products are only to be used for their intended purpose.**  
For details of all our products please visit our web site at [www.asmmedicare.com](http://www.asmmedicare.com)



### IMPORTANT NOTICE!

Please read & fully understand before using or allowing others to do so.

