

ASM

Medicare

Picow Farm Road
Runcorn, Cheshire
WA7 4UG
Tele. 01928 574301
Fax 01928 575130
E mail:-
sales@asmmedicare.com



CODE No 02053 HD

On receipt of your new heavy duty raised toilet frame please check the contents for signs of damage, report any defects to ASM immediately. The toilet frame comes fully assembled

and ready for use.

The toilet frame comes in three sizes 16", 18" & 20" This must be specified at time of ordering

The frame can be used with the pot supplied or over a conventional toilet.

To use over a conventional toilet remove the pot and place in position over the toilet

Periodic checks of all screws and fitting is recommended, replace if required.

To clean the toilet frame a mild soap and or disinfectant may be used

The intended use for this product is to provide a support aid to fit around a standard toilet or use as a commode

Maximum user weight for these products 190kgs (30stone)

NOTE:- These products are only to be used for there intended purpose. We do not recommend that the surround is used as a walking aid as this could result in personal injury.

For details of all our products please visit our web site at www.asmmedicare.com

ASM
RAISED
TOILET FRAME
USER
INSTRUCTIONS