


ASM

User
Instructions:
ASM Craven
Floor to Wall
Handrail.

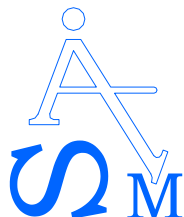

Medicare

Picow Farm Road Ind Estate
Runcorn, Cheshire. WA7 4UG.
Tel: 01928 574301
Fax: 01928 575130
E-mail: sales@asmmedicare.com

Craven Floor to Wall Handrail:

**Code: 01040W {White}
& 01040C {Chrome}.**



On receipt of your new ASM Craven handrail please check the contents for signs of damage, report any defects to **ASM Medicare Ltd** immediately. The Craven comes fully assembled and ready for use.
Please note that screws and wall plugs are not supplied.

The Craven handrails are fitted with floor and wall brackets for attaching to the floor and wall and it is recommend that a skilled tradesman or suitably competent person be employed to carry out the installation of this product.
A survey of the intended location may be required to ensure that the correct type of fixings can be identified, this will ensure your safety in the use of this product for many years to come.

If this product is to be used in a bathroom or shower room, great care should be taken to avoid slipping as the rail could become very slippery if wet or if soap gets onto the rail.

When used in a wet environment we recommend that stainless steel screws are used to avoid corrosion.
Periodic checks of *all* screws and fittings is recommended, replace if required or seek expert advise form the installer.
To clean the Craven Handrail use a mild proprietary soap and / or disinfectant. Dry with a soft clean cloth after each clean.

The intended use of this product is to provide the user with a supporting handrail.
Due to the nature of this product and the large range floors & walls which it may be fitted to, which is beyond ASM's control, a M.U.W {max user weight} is impossible to specify .

NOTE:- These products are only to be used for their intended purpose.
For details of all our products please visit our web site at www.asmmedicare.com



IMPORTANT NOTICE!

Please read & fully
understand before
using or allowing
others to do so.



Jan 2005

Better By Design