


# ASM

# Medicare

## Seville Heavy Duty Chair. Code 01102HD.

Picow Farm Road Ind Est  
Runcorn. Cheshire.  
WA7 4UG.  
Tel: 01928 574301.  
Fax: 01928 575130.  
e-mail: [sales@asmmedicare.com](mailto:sales@asmmedicare.com)

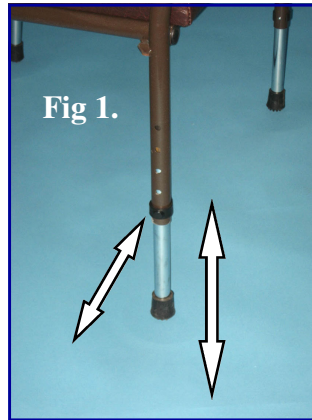

This ASM Seville Heavy Duty Chair is supplied fully assembled and ready to use, as a height adjustable chair with adjustable lumbar support for the elderly or infirm. On receipt please check for signs of damage, report any defects to ASM immediately. The chair is height adjustable, to adjust please follow the instructions below very carefully before use or allowing others to do so.

- 1} Place the chair carefully on it's side and remove the plastic clips from the two upper most legs. Slide the adjustable legs to the required position, ensuring that both legs have been set at the same height. Fig 1.
- 2} Refit the two plastic clips, ensuring that they are fully engaged. You have to twist the legs slightly in order to locate the steel pins correctly. Turn the chair over and repeat for the other two adjustable legs.
- 3} Place the chair on it's feet on a level surface & check that it is stable and does not rock, if it does , readjust the legs until it is stable.
- 4} The back support may be adjusted by loosening the two wing nuts located at the rear of the chair & sliding the back up or down to achieve the optimum position.
- 5} The seat can also be adjusted by loosening the two wing nuts located underneath & sliding the seat forwards or backwards in order to achieve the optimum position in relation to the back support. **Ensure that ALL wing nuts are fully re-tightened after each adjustment.**
- 6} Periodic checks of all screws and fittings is recommended, re-tighten or replace if necessary.
- 7} To clean the ASM Seville chair, a mild proprietary soap / disinfectant may be used. After cleaning dry with a soft clean cloth.

### NOTE:

The chair **MUST** only be used for it's intended purpose and be re-adjusted for each individual that uses it for maximum comfort. **DO NOT LAY THE CHAIR ON IT'S BACK, THIS WILL LEAD TO EXCESSIVE STRAIN ON THE ADJUSTMENT SCREWS.**

**MAXIMUM USER WEIGHTS:** Adjustable Ht: 190kg / 30st. Fixed Ht: 254kg / 40st.

ASM Seville Heavy  
Duty Chair 01102HD.

User Instructions.



### IMPORTANT NOTICE!

Please read & fully  
understand before  
using or allowing  
others to do so.



February 2007.