


ASM

Medicare

User
Instructions
ASM Step
Stool Range.

Picow Farm Road Ind Estate
Runcorn, Cheshire, WA7 4UG.
Tel: 01928 574301
Fax: 01928 575130
E-mail: sales@asmmedicare.com

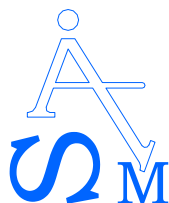


ASM Step Stool Range.

Without Handle: Code 010189.



With Handle: Code 010190.

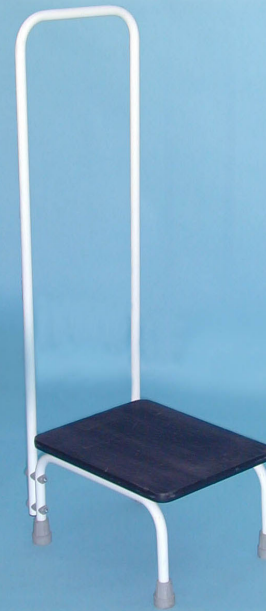

On receipt of your new ASM Step Stool please check the contents for signs of damage, report any defects to **ASM Medicare Ltd** immediately. The step stools come fully assembled and ready for use.

Please follow the instructions and guidelines below.

- 1} Place the frame on the floor as close to the item you are trying reach .
- 2} Ensure that all four legs are in contact with the floor and that the step stool is level. Failure to do so may result in personal injury.
- 3} Keep both feet central on the step stool top at all times whilst in use.
- 4} Care must be taken when using the 010190 with handle, it is *"Very Important"* that the handle is only used as a balancing aid & **not** as a weight bearing support!
- 5} Periodic checks of *all* screws and fittings is recommended, retighten or replace if required.
- 6} Clean the step stool's by using a mild proprietary soap and / or disinfectant. Dry with a soft clean cloth.

DO NOT USE THE HANDLE AS A WIEGHT SUPPORT ONLY AS A BALANCING AID!

The intended use of this product is to provide the user with aid to reaching high objects.

M.U.W = 190kg / 30st.

NOTE:- These products are only to be used for their intended purpose.
For details of all our products please visit our web site at www.asmmedicare.com



IMPORTANT NOTICE!

Please read &
fully understand
before using or
allowing others
to do so.

