

ASM

Medicare

Cork Topped Step Box.

Picow Farm Road Ind Est
Runcorn. Cheshire
WA7 4UG.

Tel: 01928 574301

Fax: 01928 575130

e-mail: sales@asmmedicare.com



Code 03030

On receipt of your new ASM Cork Topped Step Box please check for signs of damage, report any defects to ASM immediately.

The ASM Cork Step Box comes fully assembled and ready for use.
It is intended that this step box is used as an aid to getting into or out of the bath or as an aid to reach shelves or other high objects around the home.



Please note this product should only be used for its intended purpose;

- Do Not Over Reach or Stretch.
- Do Not Jump or Stamp on the step box.

As this may cause falling and lead to personnel injury.

To clean your new step box a mild proprietary soap and / or disinfectant may be used, always dry after cleaning with a soft cloth.

Do not submerge in water or soak whilst cleaning as this will result in damage to the cork top & the wood base.

M.U.W = 190kg / 30st.

For details of all our products please visit our web site at;

www.asmmedicare.com

ASM
Cork Topped
Step Box
User Instructions.



Better By Design