

ASM

User
Instructions

ASM Wren
Kitchen / Shower
Stool Range.

Medicare

Picow Farm Road Ind Estate
Runcorn. Cheshire. WA7 4UG.

Tel: 01928 574301

Fax: 01928 575130

E-mail: sales@asmmedicare.com




Codes: 02300 / 02301 / 02302 / 02303 & 02304



Wren Kitchen / Shower Stool Range.

Fig 1.

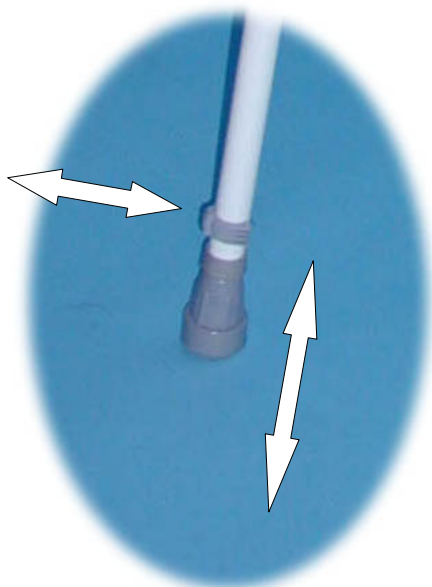

On receipt of your new ASM Wren Stool please check for signs of damage, report any defects to **ASM Medicare Ltd** immediately.

The ASM Wren Stools come fully assembled and ready for use.

With the exception of 02300, the stools are height adjustable.

To adjust please follow the instructions below.

- 1} Place the stool on it's side and remove the plastic clips from 2 legs. See Fig 1.
- 2} Slide the adjustable internal legs to the required position ensuring that both legs are adjusted equally. See Fig 1. The stools are **NOT** designed to be used with the seats level.
- 3} Replace the plastic clips, ensuring that they are **fully engaged**. You may have to twist the leg slightly to locate the steel pin correctly.
- 4} Turn the stool over & repeat for the other two adjustable legs.
- 5} Place the chair upright on it's feet on a level surface & check that it is stable and does not rock, if it does readjust the legs until it is stable.
- 6} Periodic checks of all screws and fittings is recommended, replace if required.
- 7} To clean the stool, a mild proprietary soap and / or disinfectant may be used. After cleaning dry with a soft clean cloth.

The Maximum User Weight for this product is: 152kg / 24st.

The intended use of this product is to provide the user with a "perching" type seat for those unable to sit fully.

NOTE:- These products are only to be used for their intended purpose.
For details of all our products please visit our web site at www.asmmedicare.com

Better By Design

To avoid injury please
DO NOT attempt to move
these products whilst the
patient / user is seated!

**IMPORTANT
NOTICE!**
Please read & fully
understand before
using or allowing
others to do so.

