


ASM

User
Instructions
ASM Heron
Shower Chair
Range.

Medicare

Picow Farm Road Ind Estate
Runcorn, Cheshire, WA7 4UG.
Tel: 01928 574301
Fax: 01928 575130
E-mail: sales@asmmedicare.com

Heron Shower Chair Range.

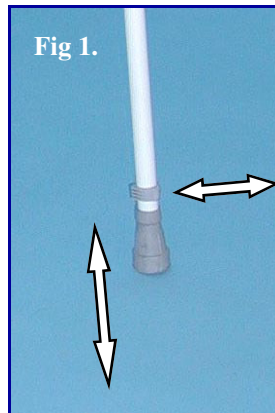

These shower chairs have been designed to assist the user who may have difficulty standing in the shower to wash independently.

There are three variants;
02164 - Static Fixed Height.
02165 - Static Adjustable Height.
2166 - Mobile Fixed Height.

The 02166 is available in three fixed height options;
16" / 18" & 20"

The M.U.W's are;
02164 - 152kg / 24st.
02165 - 127kg / 20st.
02166 - 127kg / 20st.

On receipt of your new ASM Heron Shower Chair please check the contents for signs of damage, report any defects to **ASM Medicare Ltd** immediately.

The Heron Shower Chair comes fully assembled and ready for use.

The 02165 is height adjustable, to adjust please follow the instructions below.

- 1) Lay the frame on it's side and remove the plastic clips from 2 legs. See Fig 1.
- 2) Slide the internal legs to the required position ensuring that both legs are adjusted equally. See Fig 1.
- 3) Replace the plastic clips, ensuring that they are fully engaged. You may have to twist the leg slightly to locate the steel pin correctly.
- 4) Turn the chair over & repeat for the other two adjustable legs.
- 5) Place the chair upright on it's feet on a level surface & check that it is stable and does not rock, if it does readjust the legs until it is stable.
- 6) Periodic checks of all screws and fittings is recommended, replace if required.
- 7) To clean the Heron Shower Chair, a mild proprietary soap and / or disinfectant may be used.
- 8) After cleaning dry with a soft clean cloth.

The intended use of this product is to provide the user with a Fixed Ht 02164 / Height Adjustable 02165 or Mobile Fixed Ht 02166 support aid, for sitting in a shower.

NOTE:- These products are only to be used for their intended purpose.
For details of all our products please visit our web site at www.asmmedicare.com

Better By Design

