


ASM

Medicare

Picow Farm Road Ind Estate
Runcorn, Cheshire. WA7 4UG.
Tel: 01928 574301
Fax: 01928 575130
E-mail: sales@asmmedicare.com

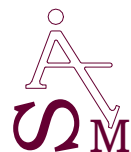

Flat Bottom Shower Chair Code No: 02020



On receipt of your new ASM Shower Chair please check for signs of damage, report any defects to **ASM Medicare Ltd** immediately. The ASM Flat Bottom Shower Chair comes fully assembled and ready for use.

- 1} Like any chair, care should be taken to avoid leaning or rocking backwards as this could lead to the chair toppling.
- 2} Do Not stand on the chair for any reason, it has not been designed for this purpose.
Both these situations could lead to personnel injury.
- 3} The drainage holes should be checked regularly and kept clear to ensure adequate drainage during use.
- 4} Periodic checks of *all* screws and fittings is recommended, replace if required.
- 5} Clean the shower chair by using a mild proprietary soap and / or disinfectant. Dry with a soft clean cloth.

The intended use of this product is to provide the user with a chair support aid, during showering.

M.U.W = 102kg / 16st.

NOTE:- These products are only to be used for their intended purpose.
For details of all our products please visit our web site at www.asmmedicare.com



IMPORTANT NOTICE!

Please read & fully understand before using or allowing others to do so.



April 2005