

# ASM


## ASM Royston Shower Chair. User Instructions



# Medicare

## Royston Shower Chair 02017.

Picow Farm Road Ind Estate  
Runcorn. Cheshire. WA7 4UG.  
Tel: 01928 574301  
Fax: 01928 575130  
E-mail: [sales@asmmedicare.com](mailto:sales@asmmedicare.com)

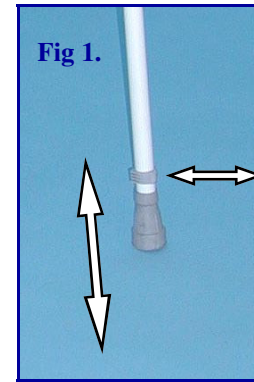

On receipt of your new ASM Royston Shower Chair please check the contents for signs of damage, report any defects to ASM immediately. The Royston comes fully assembled and ready for use.  
The Royston is also height adjustable, to adjust please follow the instructions below.

- 1} Lay the frame on it's side and remove the plastic clips from 2 legs. See Fig 1.
- 2} Slide the internal legs to the required position ensuring that both legs are adjusted equally.
- 3} Replace the plastic clips, you may have to twist the leg slightly to allow the steel pin to locate completely, it is "Very Important" that the clips are fully engaged.
- 4} Turn the frame over and repeat for the other 2 remaining legs.
- 5} Place the Royston back on it's feet on a level surface and check that it is stable and does not rock, if it does, readjust until it is stable.
- 6} Periodic checks of *all* screws and fittings is recommended, replace if required.
- 7} The plastic seat is removable for ease of cleaning and can be cleaned in the same manner as the frame, by using a mild proprietary soap and / or disinfectant. Dry with a soft clean cloth.

The mesh backrest can be machine washed at 70°.

**DO NOT TUMBLE DRY.**

**The intended use of this product is to provide the user with a height adjustable shower seat, which can be used in a shower cubical without modification.**

**The Maximum User Weight for this product is 127kg / 20st.**

**NOTE:- These products are only to be used for their intended purpose.**  
For details of all our products please visit our web site at [www.asmmedicare.com](http://www.asmmedicare.com)



### IMPORTANT NOTICE!

Please read & fully understand before use or allowing others to do so.

