

# ASM

# Medicare

Picow Farm Road  
Runcorn, Cheshire  
WA7 4UG  
Tele. 01928 574301  
Fax 01928 575130  
E mail:-  
sales@asmmedicare.com



Fig 1

Code No 02161& 02163 with back

On receipt of your new ASM Atlanta shower stool please check for signs of damage, report any defects to ASM immediately. The ASM shower stool comes fully assembled and ready for use

The stool is height adjustable, to adjust please follow the instructions below

- 1) Lay the stool on it's side and remove the plastic clips from two legs see fig 1
- 2) Slide the leg to the required position ensuring that both legs are adjusted to the same length
- 3) Replace the plastic clips, you may have to slightly twist the leg to allow the steel pin to locate correctly it is VERY important that the clip is fully engaged
- 4) Turn the stool over and repeat for the other 2 legs
- 5) Place the stool back on it's feet on a flat surface and check that it does not rock and is stable, if it does rock readjust the legs until it is stable.
- 6) If using the stool in a shower care should be taken to avoid slipping when sitting down or standing up.
- 7) Periodic checks of all screws and fittings is recommended, replace if required

To clean the stool a mild soap and or disinfectant may be used, dry off after cleaning with a soft cloth.

**The intended use of this product is to provide a comfortable and removable seat for use in a shower or bath room. It must not be used as a step and standing on it could lead to personal injury**

**MAXIMUM USER WEIGHT FOR THIS PRODUCT 152kgs ( 24 stone )**

**Note :- These products are only to be used for their intended purpose**

**For details of all our products please visit our web site at [www.asmmedicare.com](http://www.asmmedicare.com)**

ASM  
ATLANTA  
SHOWER STOOL  
USER  
INSTRUCTIONS