


ASM Medicare

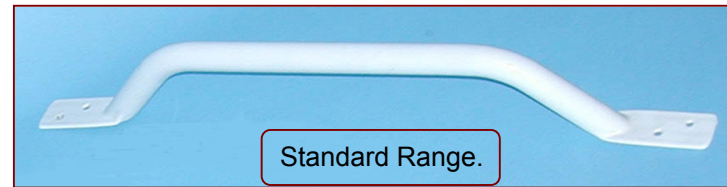

Picow Farm Road
Runcorn, Cheshire
WA7 4UG
Tel: 01928 574301
Fax: 01928 575130
Email:-
sales@asmmedicare.com

Plastic Coated Deluxe



Chrome Plated Deluxe



Chrome Plated Knurled Deluxe



This product should only be used for it's intended use.

Intended Use:- Balance support, where necessary.

On receipt of your new ASM Grabrail, please check for signs of damage & report any defects to ASM immediately.

Screws and wall plugs are not supplied with this product.

The grabrails have suitable sized holes in the foe fixing to a wall or fixture, we recommend that a suitably qualified person or skilled craftsperson be employed to carry out installation.

A survey may be required to ensure that the correct type of fixings can be identified, this will ensure your safety in the use of these products for many years to come.

The grabrails can be used in numerous situations such as, bathrooms, shower rooms, stairways, and kitchens, to name but a few. However, if the rails are to come into contact with water we recommend that our kurlled grabrail is used for extra grip, these should be fitted with stainless steel screws and sealed with silicone sealant to the wall to avoid corrosion.

To clean the rails a mild proprietry soap or disinfectant may be used.

Due to the nature of these products and the large range of walls to which they can be fitted to, which is beyond AMS's control, maximum user weight is impossible to specify. The products are fully tested to a rigorous standard which complis with the latest MDA requirements.

Note:- These products are only to be used for there intended purpose.

For details of all our products please visit our web site at;

www.asmmedicare.com

ASM
GRABRAIL RANGE
USER INSTRUCTIONS



Better By Design.