

# ASM

# Medicare

## ASM Copeland Bed User Instructions.

Overall Width: 921mm.  
Overall Lgth: 1981mm.  
Adj Platform Ht:  
406mm to 559mm.  
Overall Ht:  
952mm to 1101mm.

### IMPORTANT NOTICE!

Please read & fully  
understand before  
using or allowing  
others to do so.



January 2008.

Picow Farm Road Ind Est  
Runcorn. Cheshire.  
WA7 4UG.

Tel: 01928 574301.  
Fax: 01928 575130.


e-mail: [sales@asmmedicare.com](mailto:sales@asmmedicare.com)

### ASM Copeland Adjustable Height Bed.

Code No: 4710 Without Castors.

Code No: 4718 With Castors.

Optional Accessories Available Upon Request.

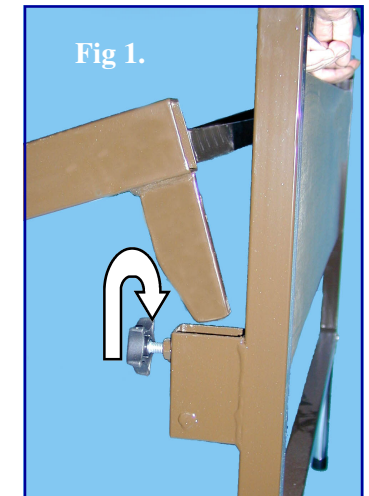

On receipt of your new ASM Copeland Adjustable Height Bed please check for signs of damage, report any defects to ASM immediately quoting the serial number.

The intended use of this product is to provide a height adjustable sleeping unit for one person.

The Copeland Adjustable Height Bed comes flat packed, to assemble please follow the instructions below.

- 1) It is advisable to adjust the height of the bed prior to assembly. This is carried out by removing the wing nut and screw from each leg and sliding the leg to the required position, replace the screws & wing nuts. Ensure that all four legs are set to the same height and that the wing nuts are fully tightened.
- 2) Lay the base of the bed on the floor with the four corner brackets facing downwards.
- 3) Fit the footboard next by placing it between yourself and the base, whilst holding the footboard with one hand reach down and lift the base up and locate the two corner brackets into the slots provided, once in position tighten the hand wheels. Fig 1.
- 4) Ensure that the corner brackets have fully entered the slots before continuing.
- 5) Repeat stages 3 & 4 for the headboard, check that all wing nuts and hand wheels are fully tightened.
- 6) The bed is now ready for the mattress to be placed in position.
- 7) Periodic checks of all screws & fittings is recommended, replace if required.
- 8) To clean the Copeland Bed a mild proprietary soap and or disinfectant may be used, dry after cleaning with a soft cloth.

Maximum User Weight = 127kg / 20st.

**Note:** These beds should only be used for their intended purpose and are not recommended for outdoor use. If the bed is to be moved to a new location we advise the user is removed first.

For details of all our other products please visit our web site at [www.asmmedicare.com](http://www.asmmedicare.com)

*Better By Design*