


ASM


Medicare

User
Instructions:
ASM Footstool
/ Legrest.

Picow Farm Road Ind Estate
Runcorn, Cheshire, WA7 4UG.
Tel: 01928 574301
Fax: 01928 575130
E-mail: sales@asmmedicare.com

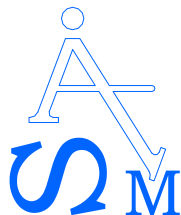

**ASM Footstool / Legrest.
Code No: 01100.**

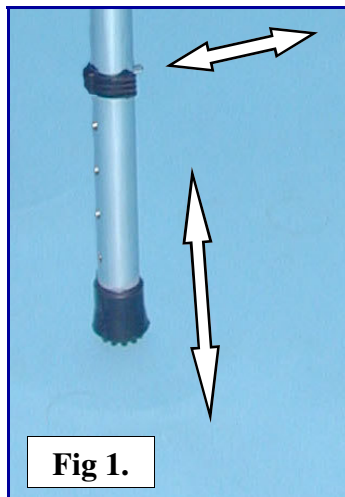

On receipt of your new ASM Footstool please check the contents for signs of damage, report any defects to **ASM Medicare Ltd** immediately. The footstool comes fully assembled and ready for use. The ASM Footstool is height adjustable for extra comfort, to adjust please follow the instructions below.

- 1} Lay the footstool on it's side and remove the plastic clips from 2 legs. See Fig 1.
- 2} Slide the internal legs to the required position ensuring that both legs are adjusted equally.
- 3} Replace the plastic clips, you may have to twist the leg slightly to allow the steel pin to locate completely, it is "Very Important" that "ALL" the clips are fully engaged.
- 4} Turn the footstool over and repeat for the other 2 remaining legs.
- 5} Place the footstool back on it's feet on a level surface and check that it is stable and does not rock, if it does, readjust until it is stable.
- 6} Periodic checks of *all* screws and fittings is recommended, replace if required.
- 7} Clean the footstool by using a mild proprietary soap and / or disinfectant. Dry with a soft clean cloth.

The intended use of this product is to provide the user with a height adjustable stool / legrest as a support aid, it is not intended to be used as a seat or step. Misuse could lead to personal injury.

M.U.W = 152kg / 24st.

NOTE:- These products are only to be used for their intended purpose.
For details of all our products please visit our web site at www.asmmedicare.com



March 2005

Better By Design