

# ASM

User Instructions  
ASM Lincoln Folding Blanket Cradle  
Code 02127.



## IMPORTANT NOTICE

Please read & fully understand before using or allowing others to do so.



March 2009.

# Medicare

Picow Farm Road Ind Est  
Runcorn. Cheshire.  
WA7 4UG.

Tel: 01928 574301.

Fax: 01928 575130.

e-mail: [sales@asmmedicare.com](mailto:sales@asmmedicare.com)

## ASM Lincoln Folding Blanket Cradle 02127.



On receipt of your new ASM Lincoln Folding Blanket Cradle, please check for signs of damage, report any defects to ASM immediately. The Lincoln Blanket Cradle comes fully assembled and ready for use. Please read and follow the instruction below.

- 1} The cradle is supplied flat, therefore it will require unfolding until it looks like the photograph above.
- 2} To fit the Lincoln, remove the top blankets or duvet and slide the larger frame underneath the mattress.
- 3} Check that there is sufficient space between the top of the cradle and the mattress for the patients legs.

If not, do not use and seek further advise.

- 4} If the space is adequate, replace the blanket or duvet and it is now ready for use.
- 5} Clean the ASM Lincoln Folding Blanket Cradle using a mild soap and / or disinfectant may be used. After cleaning dry with a soft clean cloth.

The intended purpose of this product is to provide the user with a blanket or duvet support which can be used on a standard bed to relieve the weight and contact of the blanket or duvet on the patients body and fold away when not in use.

**Note:** This product must only be used for it's intended purpose.

For details of all our products please visit our web site at;

[www.asmmedicare.com](http://www.asmmedicare.com)

*Better By Design*